



FLEXcon® Fit Test Hood

Single-use Replacement Test Hoods for Qualitative Fit Testing

PRODUCT INFORMATION

Product	FLEXcon Fit Test Hood
Contents	Single-use test hood
Indications for Use	Single-use replacement test hood for qualitative fit testing

Part Number	FLX068565
Product Dimensions	ID 24" OD 27" Length 24" Hole size 7/8" Hole Placement 12" on Center Distance from top 16"
Thickness	5 mil (0.005 inch)
Color	Clear
Material	Polyester Film
Clarity	99.7%
Haze	< 1%
Manufactured	Spencer, MA

FEATURES & BENEFITS

Disposable	Disposable single use product that prevents cross-contamination between several users.
Sturdy Structure	Optically clear PET material that provides full visibility and easy reading of test page.
Full Visibility	Opening remains in position for an easy application of the nebulizer nozzle.
Comfortable	Great for claustrophobic users as it provides 360° visibility.
Recyclable	Made of PET material that can be easily recycled.
Use	Can be used with any Respirator Fit Test kit.



Made in USA



Patent Pending

BODY CONTACTING MATERIALS

PET Film

DISCLAIMER

These single-use hoods are intended to be used as a replacement to the reusable hood that accompanies most general fit test kits. Includes the replacement hood only.



For Orders


Please visit our webpage by scanning the QR code or visit the following URL www.flexcon.com/products/test-hood

Manufactured by: FLEXcon, 1 FLEXcon Industrial Park, Spencer, MA


www.flexcon.com




FLEXcon® Fit Test Hood | Single-use Replacement Test Hoods for Qualitative Fit Testing

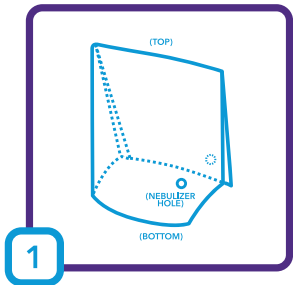


1 Orient the single-use hood so that the bottom opening is easily accessible. There is no front or back.

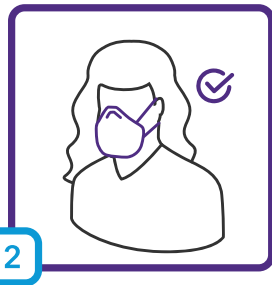

2 Ask the participant to properly apply the face mask.

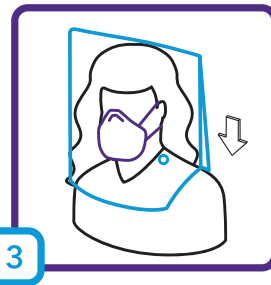


3 Open the hood and place on the participant, make sure it fits right on the shoulders.

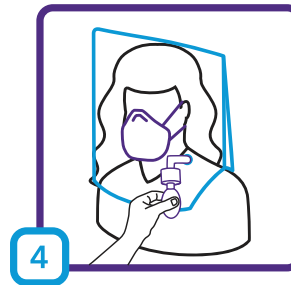

4 Place the nebulizer in the hole and disperse the solution.



5 Instruct the participant to read the story passage and check if they can detect the smell or taste of the solution.

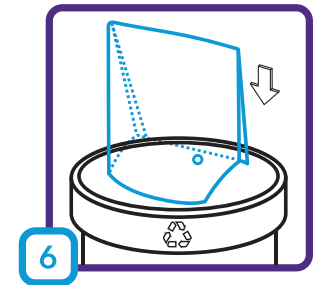

6 Upon completion of the test, remove the hood and discard in the recycle bin.



Made in USA



Patent Pending

DISCLAIMER

These single-use hoods are intended to be used as a replacement to the reusable hood that accompanies most general fit test kits. Includes the replacement hood only.



FOR ORDERS

Please visit our webpage by scanning the QR code or visit the following URL www.flexcon.com/products/test-hood